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Phone: +90 532 614 60 69

Email: info@dnamaske.com

www.dnamaske.com

6 KATLI
6 LAYERS
6 COUCHE

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APPLICATION

- 1) Remove the respirator from its packaging and hold with straps facing upward.
- 2) Fully open the top and bottom panels, bending slightly the nosepiece around your thumb at the center of the top panel.
- 3) Place the respirator on your face so that the top panel rests on your nose and the bottom panel is securely under your chin.
- 4) Pull the top strap over your head and resting high at the top back of your head. Then, pull the bottom strap over your head and position it around your neck below ears.
- 5) Adjust for a comfortable fit by pulling the top panel toward the bridge of your nose and the bottom panel under your chin. Make certain hair, facial hair, jewelry and clothing are not between your face and the respirator as they will interfere with it.
- 6) Place your fingertips from both hands at the top of the metal nosepiece. Using two hands, mold the metal nosepiece to the shape of your nose by pushing inward.

REMOVAL

- Can be performed using one or both hands.
- A) Without touching the respirator facepiece, slowly lift the bottom strap from around your neck up over your head. Then lift off the top strap. Do not touch the respirator.
 - B) Store or discard according to your facility's infection control policy.

WARNING

Respirators help protect against certain airborne contaminants. Before use the wearer must read and understand the User Instructions provided as a part of the product packaging. Misuse may result in sickness or death. For correct use, consult supervisor and User Instructions, or call DNA Maske Personal Safety Department.

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